Espiner Tissue Retrieval System

Instructions for Use: ECO-T Containment Bag
EMP400ECO-TMF-6 & EMP200ECO-TMF-6

Fannin (UK) Ltd

Yeo Bank 3, Kenn Road, Clevedon,
Somerset, BS21 6TH, United Kingdom

T: +44 (0) 1275 878 801


E: espiner@fannin.eu

W: www.espinermedical.com

Illustration A: Deployment of EMP200ECO-TMF-6 & EMP400ECO-TMF-6

15mm ID for EMP400ECO-TMF-6
12mm ID for EMP200ECO-TMF-6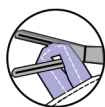

Or

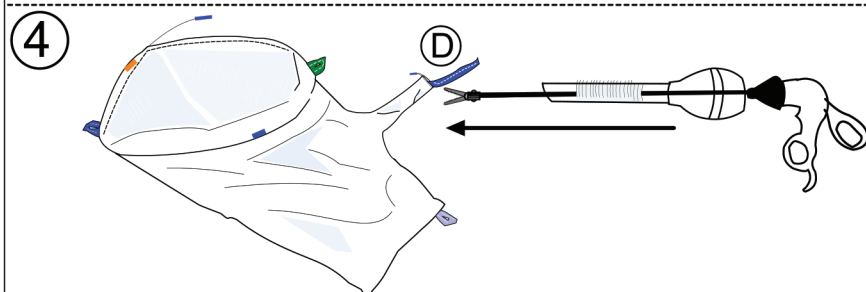
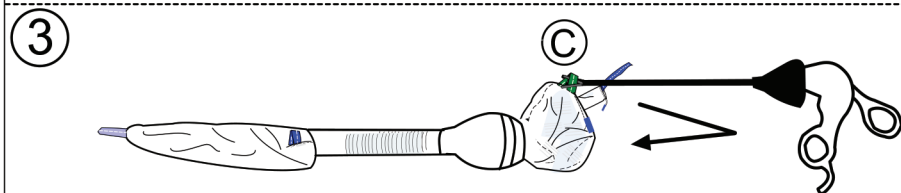
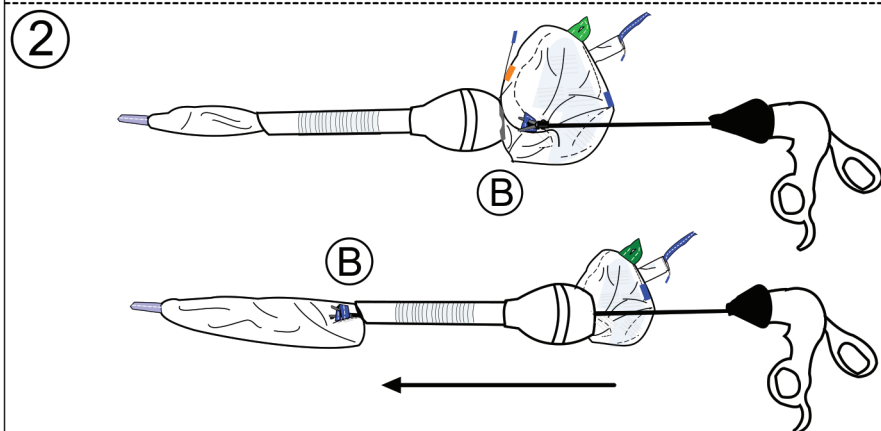
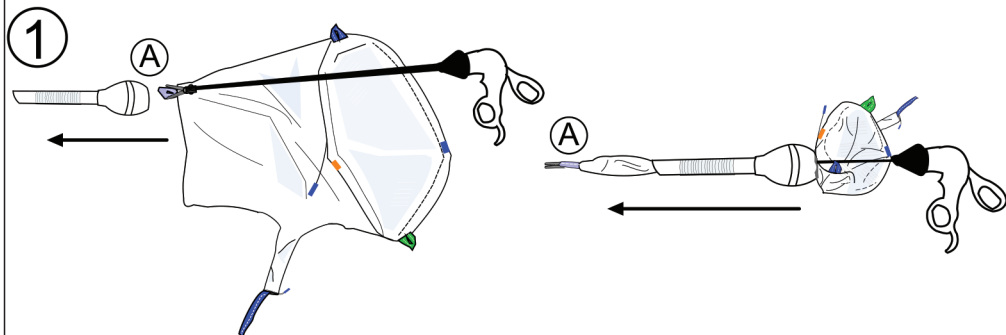
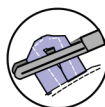
Illustration B: Tissue Capture, Use of Camera Sleeve & Morcellation

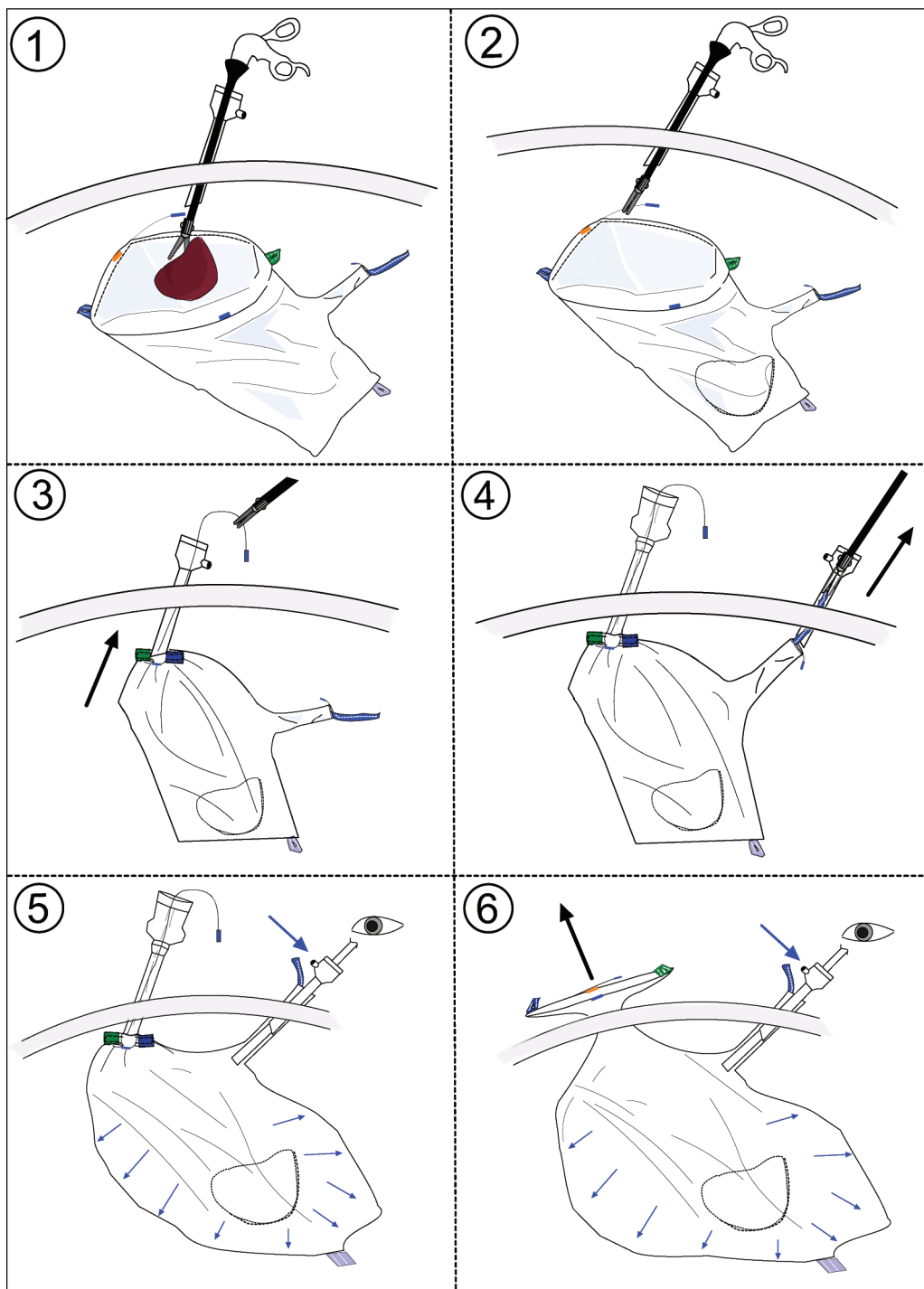
Illustration C: Morcellation


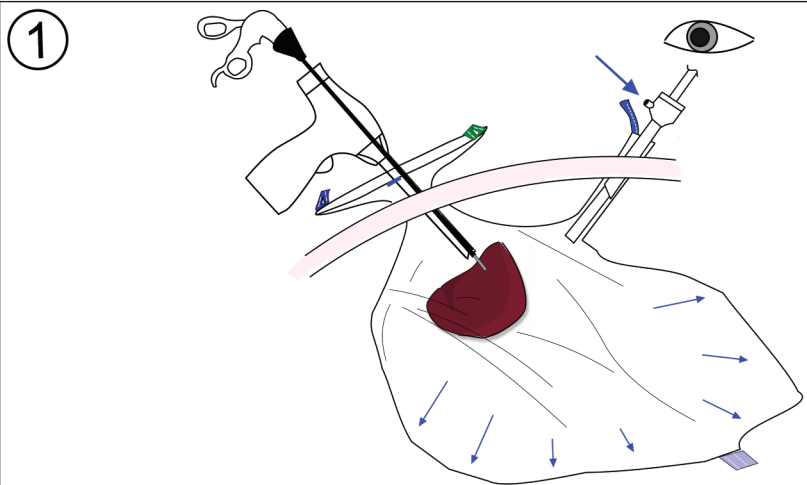
Illustration D: Withdrawal of Sac

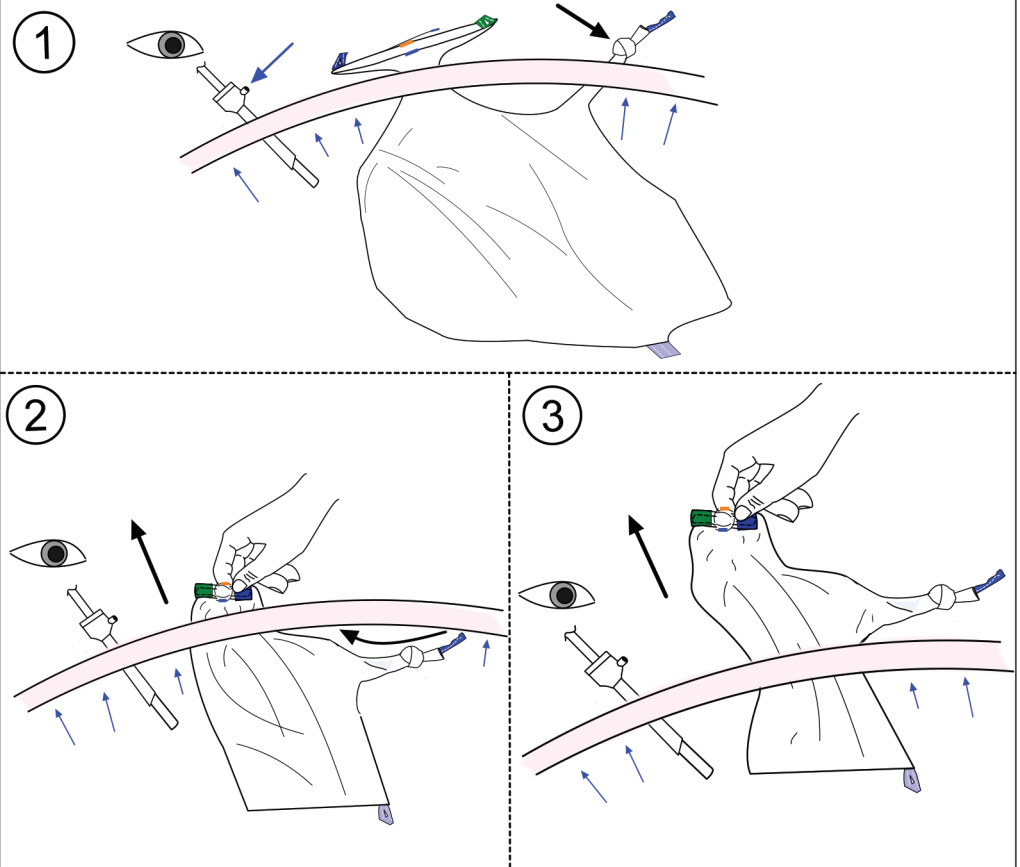Device Name: The Espiner Tissue Retrieval System

Product codes: EMP200ECO-TMF-6 & EMP400ECO-TMF-6

The ECO-TMF is a sterile, single use, tissue containment system for use in contained power morcellation minimally invasive gynecological procedures, that features an accessory sleeve as an integrated, secondary access point into the Sac. The working tip of the Morcellator is placed through the mouth of the Sac; whilst the laparoscope is inserted through the accessory sleeve, thus facilitating a multi-port approach to the isolation, containment and powered morcellation of resected tissue considered benign during laparoscopic surgery. Placing the laparoscope through the integrated accessory port on the ECO-TMF, creates a triangulation of devices, improving visualization of the morcellation procedure. Tissue containment may minimize the risk of potentially spreading the morcellated tissue. Post procedure, the sac and contents are extracted from the abdomen, and the surgeon is able to complete the procedure without the need to find lost morcellated pieces.

The ECO-TMF is intended for "single use only". The Sac is packaged in a sterile barrier (Tyvek® sealed pouch). This product is sterilized using Ethylene Oxide (ETO). The contents of an unopened, undamaged package are sterile and non-pyrogenic. Use the device prior to the "Use by Date" on the package label.

INDICATIONS FOR USE:

The ECO-TMF with integrated accessory sleeve is intended for use as a tissue containment system that enables a multi-port approach during minimally invasive gynecologic laparoscopic surgery in the isolation, containment, and removal of tissue considered benign, resected during laparoscopic surgery where contained power morcellation is performed. The ECO-TMF is compatible with electromechanical laparoscopic power morcellators that are between 15mm-18mm in shaft outer diameter and 95mm-140mm in shaft working length.

CONTRAINDICATIONS:

Do not use in gynaecologic surgery in which the tissue to be morcellated is known or suspected to contain malignancy.

Do not use for removal of uterine tissue containing suspected fibroids in patients who are: peri- or post-menopausal, or are candidates for en bloc tissue removal, for example through the Vagina or mini-laparotomy incision. (Note: These groups of women represent the majority of women with fibroids who undergo hysterectomy and myomectomy.)

Warning: Uterine tissue may contain unsuspected cancer. The use of laparoscopic power morcellators during fibroid surgery may spread cancer, and decrease the long-term survival of patients. Information regarding the potential risks of a procedure with this device should be shared with patients. The use of this containment system has not been clinically demonstrated to reduce this risk.

WARNINGS & PRECAUTIONS:

- Laparoscopic surgery should only be performed by physicians who are thoroughly trained in laparoscopic techniques, precautions and corrective actions in the event of a failure.
- Do not use after expiration date shown on label
- Check for and remove adhesions that may inhibit proper placement of the device.
- Any abdominal incision introduces a risk of abdominal hernia.
- Insert the ECO-TMF under direct vision.
- Use the drawstring by gripping the monofilament/suture below the heatshrink. The drawstring is designed for Sac closure and retrieval of the mouth from the morcellation port site. Once outside the abdomen, continue to pull on the mouth of the Sac.
- Maintain adequate insufflation of the Sac throughout the morcellation procedure, to reduce the risk of injury to internal structures and damage to the Sac. An insufflation pressure of 15-25mmHg is recommended.
- With the tip of the Morcellator in view, prior to activating the Morcellator, confirm the tissue specimen is centered within the ECO-TMF.
- When using power morcellators, the ECO-TMF should be used under direct laparoscopic vision - Do not bring the Morcellator tip into contact with the ECO-TMF.
- At all times prior to morcellating, make sure the tenaculum/grasper is within view when grasping tissue, to prevent it contacting the ECO-TMF Sac.
- When using the ECO-TMF in conjunction with powered morcellators, cutting instruments or exposed blades, care must be taken to avoid damaging the Sac.
- Do not cut, puncture or scrape the ECO-TMF with the Morcellator tip or tenaculum/grasper.
- Use tabs only for deployment of the device. Do not attempt to deploy by grasping the fabric, or by placing instruments inside the Sac.
- The ECO-TMF must be fully inflated (15-25mmHg) to minimize the risk of damage to the Sac and adjacent organs during morcellation.
- Spoon forceps should be used when only small pieces are left inside the Sac, post morcellation of tissue.
- Dispose of all used products using standard hospital practices for biohazard control.
- Do not re-sterilize. Fannin (UK) Ltd, will not be liable for any direct, indirect, incidental, consequential, or punitive damages resulting from or related to re-sterilization.
- Remove the obturator from the trocar cannula before inserting into the accessory sleeve, to avoid puncture.
- After morcellation is complete, new gloves and a new laparoscope should be used, and the old ones should be passed off the operative field.

IMPORTANT INFORMATION:

- **A 15mm Trocar is required for deployment of the EMP400ECO-TMF-6.**
- **A 12mm Trocar is required for deployment of the EMP200ECO-TMF-6.**
- This IFU is designed to assist using this product. It is not a reference for surgical techniques.
- This device is designed, tested and manufactured for single use only. Reuse, reprocessing and/or re-sterilization of this device may lead to its failure and subsequent patient infection/injury.
- All Espiner devices are not made from natural rubber latex and are sterilized using Ethylene Oxide.

The ECO-TMF is compatible with electromechanical laparoscopic power morcellators that are between 15-18mm in shaft outer diameter and 95mm – 140mm in shaft working length.

The table below indicates which sized port the ECO-TMF should be deployed through:

Sac Type	Order Code	Trocar (mm ID)	Diameter (mm)	Length (mm)	Volume (ml)	No. Per Box
EcoSac	EMP200ECO-TMF-6	12	140	325	2000	6
	EMP400ECO-TMF-6	15	320	410	4000	6

DISPOSAL

Dispose of all used products using standard hospital practices for biohazard control.

INSTRUCTIONS FOR USE - EMP200ECO-TMF-6 and EMP400ECO-TMF-6:

Illustration A: Deployment

WARNING: Insert the ECO-TMF Sac under direct vision

- 1) Using an Atraumatic Grasper, deploy Sac base tab first, by grasping the white handling tab (Tab A) at the bottom of the Sac. Grasp the white tab and insert the Sac as far as possible into the pelvis, using the full length of the graspers.
- 2) Next, grasp the blue coloured tab (Tab B) on the Sac mouth that is closest to the trocar lip and insert into the pelvis. For ease of insertion, keep the blue tab close to the rim of the trocar.
- 3) Then, remove the Grasper again and insert the green tab (Tab C) on the Sac mouth.
- 4) Lastly, grasp and insert the blue sleeve locator (Tab D). The full Sac should be within the abdomen - Refer to Illustration A, Diagram 4.

Tips:

- The coloured tabs and markers around the mouth of the Sac can be used for orientation before placement of the specimen.
- The orange marker identifies the location of the drawstring.
- The green and blue tabs identify the lateral edges of the Sac, with the green tab identifying the side with the accessory sleeve
- The black stitching around the mouth, indicates the inside of the sac mouth.

Illustration B: Tissue Capture, Use of Camera Sleeve & Morcellation

- 1) Place the specimen inside the ECO-TMF, using the black stitching to locate the opening of the Sac mouth.
- 2) Once the specimen is securely within the Sac, pull the drawstring using graspers to close the Sac mouth, capturing the tissue.
- 3) Continue to extract the drawstring through the morcellation port site.
- 4) Locate the blue sleeve locator and with a grasper, extract the accessory camera sleeve from the abdomen, through the port site where the laparoscope will be inserted into the Sac.
- 5) Place a 5-10mm Trocar inside the accessory sleeve.

WARNING: Remove the obturator from the trocar to avoid puncturing the Sac.

Allow desufflation of the abdomen from another trocar (not within the Sac) and leave that port open, then re-attach the insufflation to the 5-10mm Trocar within the accessory sleeve.

WARNING: An insufflation pressure of 15mmHg-25mmHg is recommended.

The camera is then inserted through the trocar to confirm placement inside the Sac.

- 6) Remove the 12/15mm Trocar and use the drawstring to extract the mouth of the Sac from the abdomen.

WARNING: The drawstring is designed for Sac closure and retrieval of the mouth from the morcellation port site. Once outside the abdomen, continue to pull on the mouth of the Sac. Open the mouth of the Sac ready for the introduction of the Morcellator.

Illustration C: Morcellation

WARNING: When using power morcellators, all contents within the ECO-TMF Sac should be under direct laparoscopic vision.

- 1) When insufflation is satisfactory, insert the Morcellator under direct vision and begin Morcellation.

WARNING: An insufflation pressure of 15mmHg-25mmHg is recommended.

WARNING: Spoon forceps should be used when only small pieces are left inside the Sac, post Morcellation of tissue.

Illustration D: Withdrawal of Sac

- 1) After Morcellation, the insufflation tubing should be moved to the contralateral trocar not within the Sac and the abdomen re-insufflated to 15mmHg. Remove the laparoscope and change to new gloves and laparoscope to avoid risk of contamination. Remove the trocar from the accessory sleeve. Secure by tying a knot, as close as possible to the distal end of the accessory sleeve and as tight as possible. The blue sleeve locator can be used to aid in securing the knot.
- 2) Remove the Sac and remaining contents, by inserting the new laparoscope through the alternate 5mm trocar (where the insufflation tubing is attached), to observe Sac removal under direct visualization. Pull the mouth of the Sac until the accessory sleeve is pulled back into the abdomen.
- 3) Continue to pull Sac mouth, using a 'rocking motion' under direct vision, until Sac and contents are clear of patient.

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