




# The Espiner Tissue Retrieval System

EcoSacs 70 100 140


## Instructions for use

Deploy mouth first

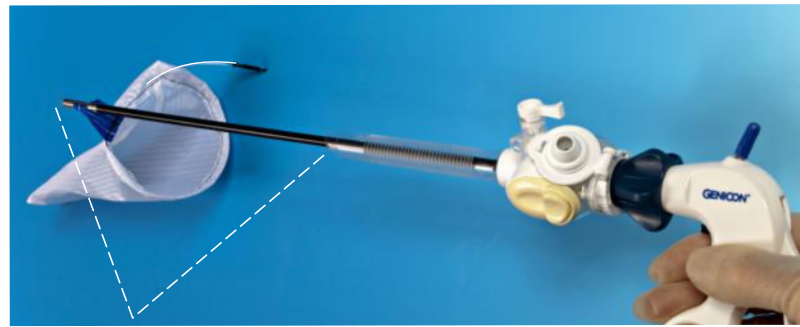


Take care to be sure there is free space in front of the grasper

Deploy bottom first

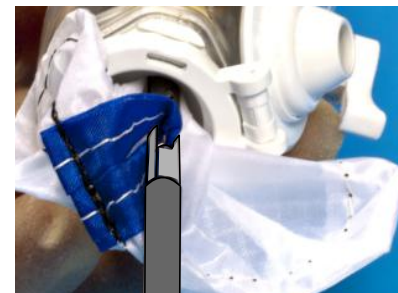

Use the grasper to take the tab over the outer rim of the the mouth and down the trocar

Close mouth and remove

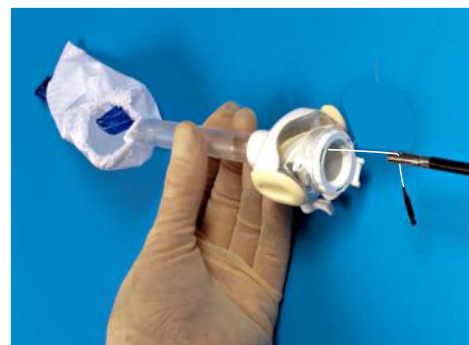

Do not use instruments inside the sac for deployment. Use the tabs

Caution: Federal (USA) and European Union law restricts this device to sale, distribution and use by or on the order of a physician.



The device is only to be used by Surgeons who are experienced with the specific techniques required by these medical devices.