




Espiner Medical Ltd

Using the Master E-Sac


Before using the E-sac make sure you know which way is "UP"
The black stitching is *inside* the mouth
The top side of the ring is embossed with *UP*

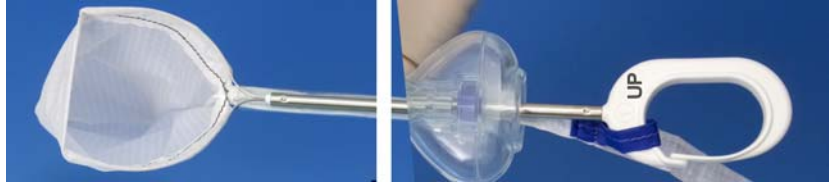

To load the E-Sac into the delivery tube simply pull on the ring as shown



As the sac is loading, squeeze the trapped air to ease the entry

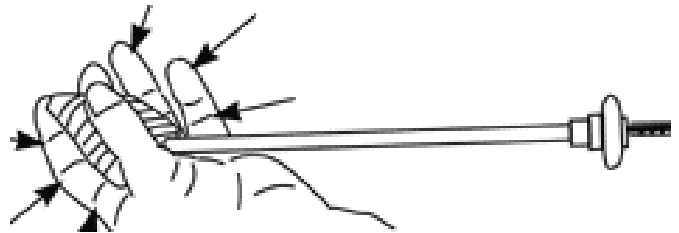

Stop pulling just before the tip of the sac disappears




The assembly is now ready to use - but make sure the ring at the back of the rod is facing *UP*


Pass the assembly into the operating port cannula



Deploy the sac by pushing the rod fully home.
The mouth opens automatically as it exits
the tube

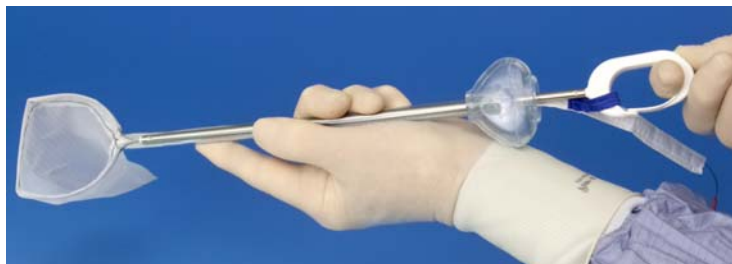

Once the tissue is captured, disconnect
the sac from the rod by releasing
the blue loop from the ring

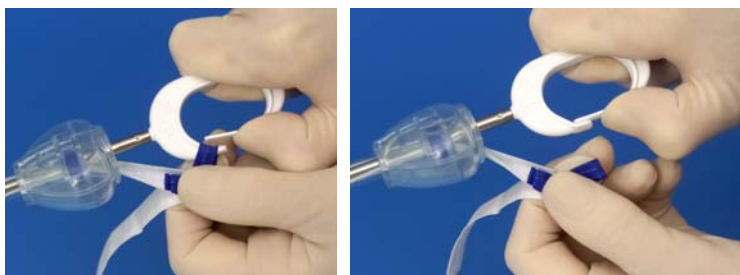

The rod is removed leaving the sac free.
An instrument can be passed through
the valved tube to assist in extraction



Pull the drawstring
at the end of the tail
to close the mouth
of the sac



Extraction is completed by removing the tube and port cannula over the tail of the sac
and pulling the mouth up through the wound. The drawstring is easily opened to give access
To the contents for morcellation

Alternatively the tail may be passed back through the tube and port cannula
to be withdrawn through another port site more suitable for extraction