

Description

Developed in close association with colorectal surgeons and theatre nurses, the **Proctowash™** is a quick, sealed, easy-to-use kit for washing out the rectum and distal colon. The device comprises:

- A** Probe with flexible tip for atraumatic insertion without obturator; sealed inlet port for irrigation, and sealed outlet port for waste/drainage
- B** 2.5m fluid administration set with atraumatic tip
- C** Large bore waste tubing
- D** Detachable, resealable 3-litre container
- E** Sealing cap

How to Use

- 1** Insert the probe into the anus and feed the fluid administration set through the inlet port, along the probe and into the rectum.


There is sufficient length of tubing for advancing further into the colon if desired.
- 2** Spike a bag of appropriate irrigation fluid and irrigate. A topical solution may be added.
- 3** Effluent drains through holes in the nozzle of the probe and down the waste tubing to the container.
- 4** When finished, detach container, seal with cap provided and discard.

Quick, Clean, Convenient

For further information, please contact Intermark on **020 8467 3355**.



Order Code: Proctowash PR 01

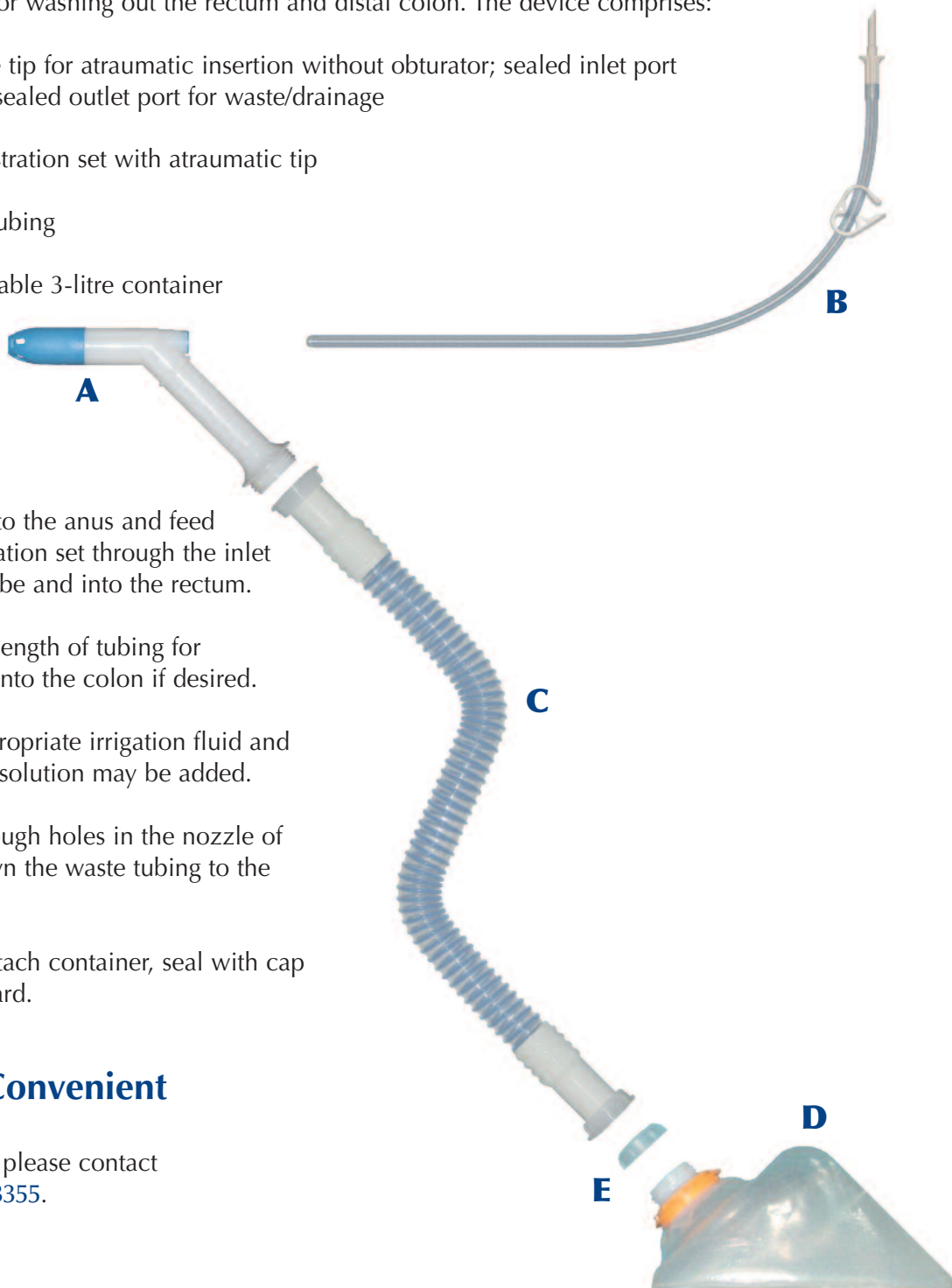

INTERMARK MEDICAL INNOVATIONS LTD

The Coach House
34 Southborough Rd
Bromley
Kent BR1 2EB
UNITED KINGDOM

UK Tel: 020 8467 3355
UK Fax: 020 8467 3364
Int. Tel: +44 20 8467 3355
Int. Fax: +44 20 8467 3364
e-mail: info@intermarkmedical.com